



Keratin and Futura

Washing/Conditioning

- Gently brush the cranial prosthetic to remove any teasing or tangles. When brushing, start at the ends and work your way to the roots.
- Swish gently in cool or lukewarm water using recommended cleaning solution. Rinse thoroughly in cool or lukewarm water. Lightly squeeze hair, making sure not to rub hair.
- Gently apply a small amount of recommended cleaning solution to the hair and leave it on for about 3 minutes. Rinse thoroughly in cool lukewarm water. Lightly squeeze hair, making sure not to rub the hair.
- Gently blot hair with towel.
- Comb wet hair, one section at a time, beginning at the ends of the hair and moving your way up using gentle downward strokes. Never begin combing at the top or middle and always comb down.

Drying

- Allow hair to dry naturally or use medium heat flow in dryer setting.
- Hang up or place cranial prosthetic on a stand and allow it to dry at room temperature.

Styling

- Always use a wide tooth or pick comb for curly styles and a wire brush for straight styles. Avoid standard brushes.
- When brushing, start at the ends and gently work your way to the roots.
- To completely restyle your human hair cranial prosthetic, we suggest taking it to your stylist. If you decide to restyle it yourself please follow these guidelines:
 - Place cranial prosthetic stand. Use standard or electric rollers or pin curls.
 - Remove tangles with a wire brush or wide tooth comb.
 - Pick up the hair and wind a roller, smoothing the ends as you go.
 - Use a medium setting with electric rollers or hair dryer. When blow drying, keep 12" from the hair dryer while circulating heat evenly and continuously.
 - Allow hair to cool before removing rollers or pin curls.
 - Lightly brush or tease desired areas and spray with recommended styling product.



Synthetic

Washing/Conditioning

- Gently brush cranial prosthetic to remove any teasing or tangles. When brushing, start at the ends and work your way to the roots.
- Swish gently in cool or lukewarm water using recommended cleaning solution. Rinse thoroughly in cool or lukewarm water. Lightly squeeze hair, making sure not to rub hair.
- Gently apply a small amount of recommended cleaning solution to the hair and leave it on for about 3 minutes. Rinse thoroughly in cool lukewarm water. Lightly squeeze hair, making sure not to rub the hair.

Drying

- Gently towel blot hair. Hang up or place on stand and allow to air dry at room temperature.
- Do not comb or brush when cranial prosthetic is wet. When completely air dry or cool dryer setting, brush or use a pick comb to achieve desired style. No heat styling!

Styling

- For tight curls or ringlet styles, try the “Shake and Go” method, or gently lift and separate the curls using your fingers or a pick comb.
- For wavy styles with loose curls, brush lightly with a styling brush or a pick comb, using short lifting strokes.
- For long curly styles, use your fingers or a pick comb to gently separate the curls.
- For straight styles, brush using long, even gentle strokes. Starting at the bottom, gently brush the hair using a downward motion and gradually move up.
- Never begin combing at the top or middle; it will tangle and damage the hair.
- We do not recommend wearing during sleep.
- NEVER expose your cranial prosthetic to heated styling tools such as hair dryers, curling irons or other heat sources.

***ALWAYS HELPFUL TO HAVE A CONSULTATION WITH A
COSMETIC HAIR EXPERT***